

Retractable Disposable Scalpel - Sterile

ABS Handle fitted with Stainless Steel Surgical Blade, individually packed

Product	Product No	Unit	Product	Product No	Unit
10 scalpels individually packed			25 scalpels individually packed		
10	4901	Box of 10	10	3901	Box of 25
10A	4902	Box of 10	10A	3902	Box of 25
11P	4991	Box of 10	11P	3991	Box of 25
12	4904	Box of 10	12	3904	Box of 25
14	4919	Box of 10	14	3919	Box of 25
15	4905	Box of 10	15	3905	Box of 25
15C	4921	Box of 10	15C	3921	Box of 25
20	4906	Box of 10	20	3906	Box of 25
21	4907	Box of 10	21	3907	Box of 25
22	4908	Box of 10	22	3908	Box of 25
23	4910	Box of 10	23	3910	Box of 25
24	4911	Box of 10	24	3911	Box of 25
25A	4915	Box of 10	25A	3915	Box of 25
Stitch Cutter	4926	Box of 10	Stitch Cutter	3926	Box of 25
10R	4990	Box of 10	10R	3990	Box of 25

Retractable Disposable Scalpel - Non-Sterile

ABS Handle fitted with Stainless Steel Surgical Blade, bulk packed

Product	Product No	Unit	Product	Product No	Unit
500 scalpels bulk packed			500 scalpels bulk packed		
10	4001	Box of 500	21	4007	Box of 500
10A	4002	Box of 500	22	4008	Box of 500
11P	4091	Box of 500	23	4010	Box of 500
12	4004	Box of 500	24	4011	Box of 500
14	4019	Box of 500	25A	4015	Box of 500
15	4005	Box of 500	Stitch Cutter	4026	Box of 500
15C	4021	Box of 500			
20	4006	Box of 500	10R	4090	Box of 500

Swann-Morton and the 'ring pattern' logo are registered trademarks of Swann-Morton Limited and related companies. The products shown are manufactured for use by trained health care professionals and can cause injury if used or handled incorrectly. The information and illustrations used in connection with the products shown on this leaflet are for general guidance only and users must seek proper training and instruction in their use before using or handling such products. Swann-Morton Limited exclude all liability for injury to the extent permitted by law.

Swann-Morton®
Sheffield - England

Swann-Morton Ltd. Penn Works, Owlerton Green, Sheffield S6 2BJ
Telephone: 0114 2344231, Sales 0114 234422
uksales@swann-morton.com, exportsales@swann-morton.com
orders@swann-morton.com, www.swann-morton.com

RET4pp/issue 2 03/20

Finger on the Trigger

Extend to Use and Retract for Safety


Retractable Safety
Scalpels by

Swann-Morton®




An extended Retractable Scalpel range with Sharps Safety compliance in mind.

The EU Council Directive 2010/32/EU accepted into UK law under the Health and Safety (Sharps Instruments in Healthcare) regulations 2013 aims to achieve the safest possible environment for Healthcare workers through the prevention of sharps related injuries. Medical devices incorporating engineered safety mechanisms and protection should be "considered" by staff as "part" of their risk assessment process alongside training and education in best practice when handling sharps. The Swann-Morton retractable safety scalpel is one such device and is now available in an extended range offering 15 blade shapes including a Safety Stitch Cutter.

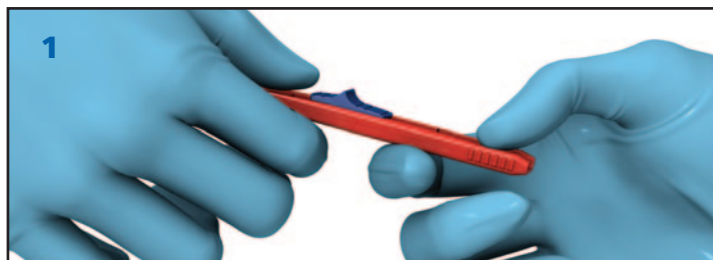


1 The high visibility red handle body contains a mid-safe passing notch into which the coloured slider is located whilst retracted so that the blade is held securely and safely when passing between members of the surgical team.

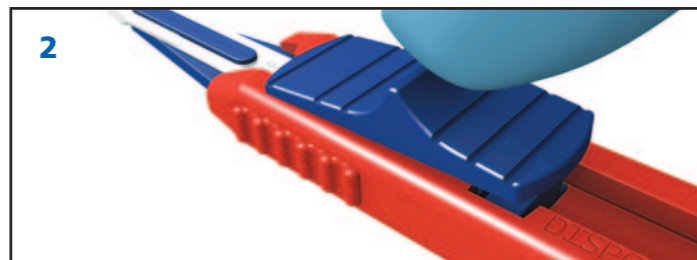

2 The user can expose the blade by placing downward pressure onto the sliding trigger and pushing it forward to engage the temporary usage position.



3 This forward lock is strengthened to prevent any uncontrolled retraction of the blade when coming into contact with hard surfaces or bone during use.

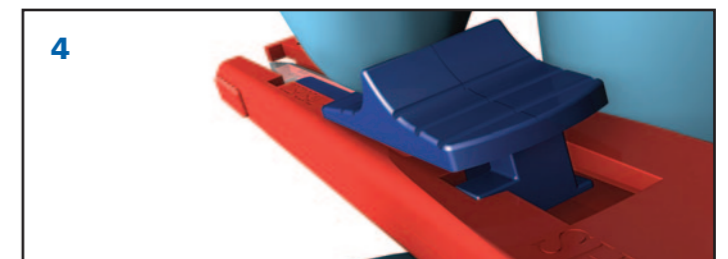

4 Once the procedure is finished and the count completed the trigger should be moved from the safe passing position towards the back of the handle body, where with a conscious amount of effort, it will engage the permanent locking position ready for final disposal in a sharps container.



2797



EMS 633531



ENMS 633525



FM 73368



Scan this code for more product information