

RECOMMENDED PRODUCT FAMILIES FOR USE IN PROCEDURE/CUSTOM KITS AND TRAYS

The following product families are recommended for use in procedure/custom kits and trays:

PRODUCT FAMILY	PRODUCT REF PREFIX CODE	GAMMA	ETO
STERILE CARBON STEEL SURGICAL BLADES Unit Pack Metal foil packet	02	Suitable	Suitable*
STERILE STAINLESS STEEL SURGICAL BLADES Unit Pack Metal foil packets	03	Suitable	Suitable*
NON-STERILE DISPOSABLE SCALPEL WITH GUARD Unwrapped Bulk Packed	23	Suitable	Suitable
NON-STERILE RETRACTABLE SCALPEL Unwrapped Bulk Packed	40	Suitable	Suitable
NON-STERILE BLADE REMOVER Unwrapped Bulk Packed	5503 5505	Suitable	Suitable

*Due to the pressures changes involved during EtO sterilization the seal on the foil packet may weaken as a result. We would recommend that products only be subjected to one cycle.

The following product families can be included as part of a Procedure/Custom Kit or Tray however sterilization via EtO process may compromise the product packaging.

STERILE DISPOSABLE SCALPEL Unit Pack clear film & tray	05	Suitable	Due to the pressure changes involved the packet seal may become weak. The scalpel may also move and puncture the packet
STERILE RETRACTABLE SCALPEL STERILE KLEEN BLADE MANAGEMENT SYSTEM Unit Pack paper & clear film	39 57	Suitable	Suitable (depending on parameters used). Due to the pressure involved in this process the packet seal may be breached
STERILE BLADE REMOVER Unit pack paper & clear film	5502	Suitable	Due to the pressure involved in this process the packet seal may be breached
STERILE DISPOSABLE SCALPEL WITH GUARD Unit Pack paper & clear film	66	Suitable	Due to the pressure involved in this process the packet seal may be breached

Steam Autoclave is not recommended for any of the above products. Please note that the packaging identified above for sterile products is not gas permeable.



Darren Hall
QA/RA Systems Manager
10th August 2020