


For use within rigorous Orthopaedic and General surgical procedures.

Cygnetic® provides the surgeon with additional strength and stability through the patented combination of its specially designed blade and handle.

Its functionality will assist the nurse with safe and easy blade attachment and removal whilst the simple disassembly for cleaning will be welcomed by the decontamination department.

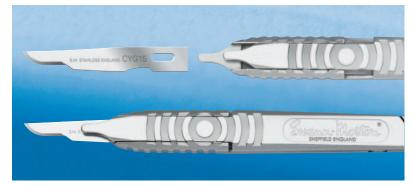

Easy to attach, easy to detach and easy to clean.


















ONE OF THE WORLD'S LEADING SURGICAL BLADE MANUFACTURERS

Cygnetic[®] by Swann-Morton

For use within rigorous Orthopaedic and General surgical procedures.

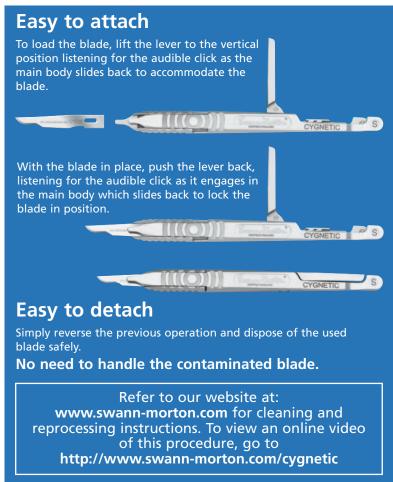



To reassemble place the lever into the 'half moon' indentation, located on the graduated bottom section, identified with the Cygnetic® mark. The lever should be in the forward position. Take the top section and slide it into the forward guides of the bottom section while at the same time moving the lever back to the vertical and placing gentle downward pressure on the Swann-Morton logo. Listen as the 3 parts click into place. Continue to move the lever back towards the handle body to lock the component parts into place. The handle is now ready to use for the next procedure.



Product	Product No	Unit
Cygnetic® Range Blades - Sterile Blades in Stainless Steel - Sterile, individually V.P.I. protected in Metal Foil Packets		
10	5301	Box of 50
11	5303	Box of 50
15	5305	Box of 50
20	5306	Box of 50
Cygnetic® Range Handles - Non-Sterile Handles in Stainless Steel - Non-Sterile, individually sleeved		
Cygnetic [®]	5398	1

Swann-Morton and the 'ring pattern' logo are registered trademarks of Swann-Morton Limited and related companies. The products shown are manufactured for use by trained health care professionals and can cause injury if used or handled incorrectly. The information and illustrations used in connection with the products shown on this leaflet are for general guidance only and users must seek proper training and instruction in their use before using or handling such products. Swann-Morton Limited exclude all liability for injury to the extent permitted by law.

